



WeightRight™ Mercury-Free Bougies (1208-, 1209-series)
Traditional™ Mercury-Filed Bougies (1210-, 1212-series)

Updated Sterilization and Cleaning Instructions

The bougies can be sterilized with the Steris System 1E® or the STERRAD NX® System. The STERRAD NX® sterilization is based on a standard 28 minute cycle. The maximum load limit for the STERRAD NX® System is 4850g or 10.7lbs per shelf.

The bougies are compatible with Cidex® OPA High Level Disinfectant, Steris Resert™ XL HLD High Level Disinfectant and the Reliance™ EPS Endoscope Processing System.

Please refer to the manufacturer's instructions for complete information and precautions.

The bougies are also compatible with the Medivators Advantage Plus system, used with Rapicide PA High Level Disinfectant, from Minntech Corp.

Please refer to the manufacturer's instructions for complete information and precautions.

medovations, inc.
102 east keefe avenue
milwaukee, wi 53212 usa
800.558.6408
414.265.7620
Fax 414.265.7628
medo@medovations.com
www.medovations.com

Addendum to 063-0006, 063-0008, 063-0026, 063-0027
ECO1419
9-3-2010



Safeguarde™ Wire Guided Dilators (1214-series)

Updated Sterilization and Cleaning Instructions

The dilators can be sterilized with the Steris System 1E®.

The dilators are compatible with Cidex® OPA High Level Disinfectant, Steris Resert™ XL HLD High Level Disinfectant and the Reliance™ EPS Endoscope Processing System.

Please refer to the manufacturer's instructions for complete information and precautions.

The dilators are also compatible with the Medivators Advantage Plus system, used with Rapicide PA High Level Disinfectant, from Minntech Corp.

Please refer to the manufacturer's instructions for complete information and precautions.

medovations, inc.
102 east keefe avenue
milwaukee, wi 53212 usa
800.558.6408
414.265.7620
Fax 414.265.7628
medo@medovations.com
www.medovations.com

Addendum to 063-0006, 063-0008, 063-0026, 063-0027
ECO1419
9-3-2010



M-Flex™ Blue Silicone Bougies (1206-, 1207-series)

Updated Sterilization and Cleaning Instructions

The bougies can be sterilized with the STERRAD NX® System. The STERRAD NX® sterilization is based on a standard 28 minute cycle. The maximum load limit for the STERRAD NX® System is 4850g or 10.7lbs per shelf.

The M-Flex™ silicone bougies can also be sterilized by autoclave using a standard sterilization cycle of 132°C for 5 minutes. M-Flex™ Bougies sterilized by steam sterilization machines should be individually wrapped.

These bougies are compatible with Cidex® OPA High Level Disinfectant, Steris Resert™ XL HLD High Level Disinfectant and the Reliance™ EPS Endoscope Processing System. Please refer to the manufacturer's instructions for complete information and precautions.

The bougies are also compatible with the Medivators Advantage Plus system, used with Rapicide PA High Level Disinfectant, from Minntech Corp.

Please refer to the manufacturer's instructions for complete information and precautions.

medovations, inc.
102 east keefe avenue
milwaukee, wi 53212 usa
800.558.6408
414.265.7620
Fax 414.265.7628
medo@medovations.com
www.medovations.com

Addendum to 063-0006, 063-0008, 063-0026, 063-0027
ECO1419
9-3-2010